


# PAL™ - Plastic Articulator Lite Instructions

## *Creating a functional link between disposable and semi-adjustable articulators!*

- Anatomical Axis of Rotation  
(same as the full size articulator)
- Horizontal and Vertical Guides
- Built-In Support Stand

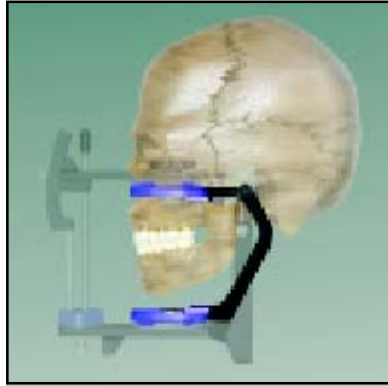

- Newly Designed Mounting Plate  
One Index for PAL...  
Another for Articulator
- Disposable or Semi-Reusable
- Open 100° to separate frames



### Instructions:



1. Mount study casts to magnetic mounting plates on the full size (model PCH, PSH) Panadent articulator using the Kois Dento-Facial Analyzer or the traditional Pana-Mount Face Bow and an interocclusal record.



2. Remove study casts attached mounting plates from the full size Panadent articulator. Clip in the new Panadent PAL articulator into the receiving port located in the back of the plastic magnetic mounting plate.



3. After performing laboratory procedures of making restorations, the PAL articulator can be detached from the study casts by pressing the release button located on the bottom of the plastic magnetic mounting plate.



4. The study casts can now be placed back on the full size Panadent articulator so that the restorations can be quality controlled by using the guidance of the curvilinear motion analogs and perform any final adjustments if required.