

Centric Latch Replacement Instructions:

Items necessary to replace the Centric Latch:

- 1 #2023 Centric Latch
- 1 #9702 Latch Spring
- 1 #9906 Hex Wrench or (1/16" Allen Wrench)
- 1 #9904 Hex Wrench For Articulator or (3/32" Allen Wrench)

Centric Latch Removing :

Loosen thumb screws on blue Analogs to remove Dyna-Links from Analogs. Release Centric Latch and remove upper articulator frame from lower frame. Holding articulator, using articulator hex wrench loosen and remove Centric Channel screw. Remove brass Centric Channel. Using a 1/16" hex wrench, loosen set screws of Centric Latch hinge pins. Push Centric Latch hinge pins out by accessing through Centric Channel slot. Remove Centric Latch. Note: Be careful not to loose Centric Latch Spring

Centric Latch Placement :

Place new centric latch into lower frame of articulator. Push in hinge pin on other side just until it starts into Centric Latch. Place and push spring into position of Centric Latch, then push in hinge pin. Tighten Centric Latch hinge pin set screws. Replace upper articulator frame and adjust Centric Channel as necessary.