

Magna-Split II System Instructions



Fasten Metal Mounting Plate to Articulator with fastening screws.




Attach Magnet to Mounting Plate with mounting plate screw.



Place plastic index plate to metal magnetic mounting plate.



Add plaster to plastic index plate and study cast to mount models in usual manner.

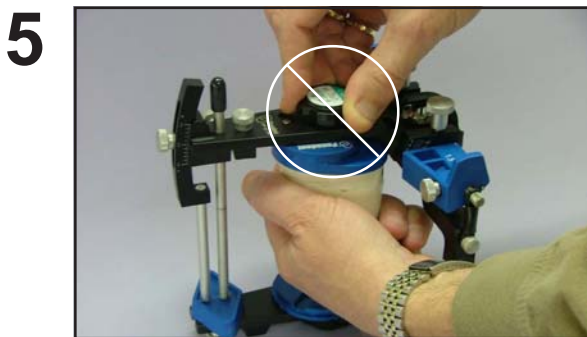

To prevent losing the magnet assembly, Do not try to remove your casts by unscrewing the knob, the magnet housing and magnet will fall out or stay with the mounted casts.



To prevent casts from separating from mounting plate, grasp the cast and mounting plate with thumb and fingers. Tilt the mounted cast assembly sideways to release cast from mounting plate magnet. (Do not pull on mounted casts.)

Check to make sure your magnet is still on your articulator, without a magnet, your stone transfer plates will not connect.



⊗ 2860: If used more than one time, insert may come loose and plate will no longer index to base plate.