



Panadent

■ Panadent Corporation
580 S. Rancho Avenue • Colton, California 92324, USA
Tel: (909) 783-1841 • USA & Canada (800) 368-9777

REF 1610,1620,
1650, 1701,
1801, 1901

ARTICULATOR CARE AND MAINTENANCE

Your Panadent articulator is a precision instrument which requires care and periodic maintenance. Do not subject articulator to extreme temperatures, pressures or conditions as it may cause articulator to become out of factory specifications as well as effect the finish.

CLEANING

It is recommended to use silicone spray to clean and wipe away accumulations of wax, porcelain or carborundum grit.

LUBRICATION

Apply thin coating of Petroleum Jelly with cotton swab periodically to condylar guiding area of Motion Analogs. Lightly spray entire surface of both upper and lower frames with a thin film of silicone spray each time before mounting casts so that mounting stone can be easily removed from instrument with fingers only.

STORAGE

When not in use, store articulator in a clean, dry atmosphere. DO NOT store near acids, alkalies or medicaments- since their fumes may corrode instrument.